


DTX Studio Clinic Activation in an Offline Environment

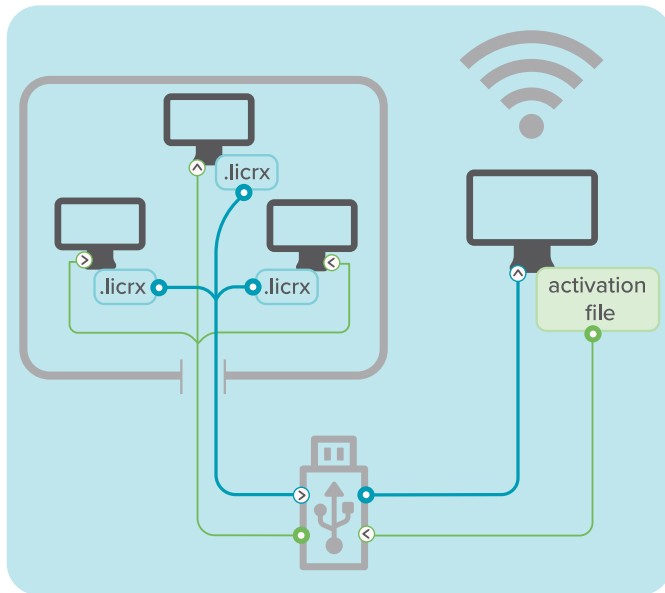

! **Important**

Offline deployment is permitted **only in exceptional cases** where the customer's IT environment prevents standard online deployment. A dedicated **DTX Studio Clinic Offline license is required** and the **Commercial Vice President's approval is mandatory** before this license can be granted.

- Access to **at least one** computer with an **active internet connection** is required.
- Perform offline activation **for each computer** separately.
- You can collect **multiple** computer identifier files on a **single USB device**.
- On the **connected** computer, generate **one activation file** that includes all selected computers.
- Download the **installation files** and **store on a USB stick** or similar external storage.

1

1. Install the software on the offline computer.
2. In the **Configuration wizard**, accept the license agreement terms and click **Next**.

2

Select **No internet? (offline activation)** below the installation key input fields.

3

Click **Create File** to create the computer identifier file.


4

1. Browse to the folder where you want to store the computer identifier file and click **Select folder**.
2. Copy the identifier file (.licrx) to a USB device.

Note


If multiple PCs require installation, repeat steps 1 to 4 to collect the identifier files for all PCs on which DTX Studio Clinic will be installed.

5

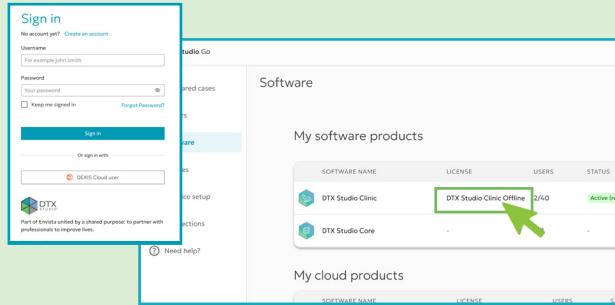


1. Insert the USB device into a computer with an internet connection, go to <https://go.dtxstudio.com> and sign in.
2. Select **DTX Studio Clinic Offline** on the **Software** tab.

6

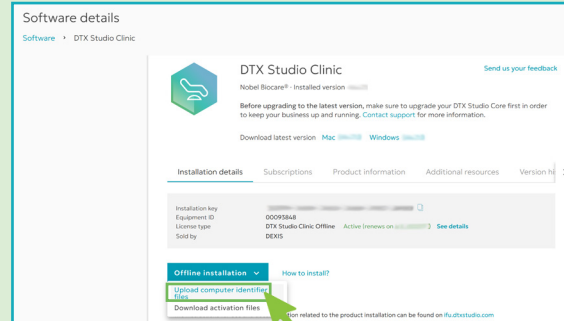


1. Select **Offline installation**.
2. Choose **Upload computer identifier files** from the drop-down list.

7

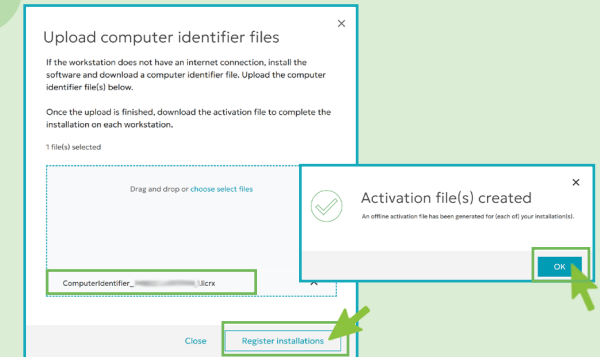


1. Select the .licrx files or drag and drop them to upload the files.
2. Click **Register Installation**.
3. Click **OK** to continue.

8

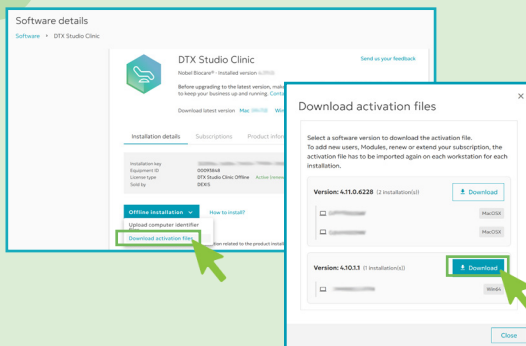

1. Select **Offline installation** and choose **Download activation files** from the drop-down list.
2. Then click **Download** in the **Download activation files** window and **Close** when the download is done.
3. **Copy the activation file** to the USB device.

9

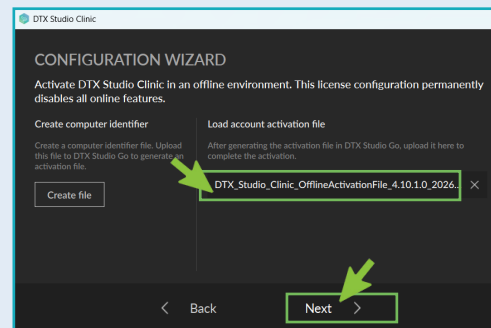

1. Return to the offline computer and load the activation file in the **Configuration wizard** (under **Load account activation file**). Select the file or drag and drop it.
2. Click **Next**.

10

Follow the further steps in the **Configuration wizard** and click **Finish** to finalize the installation.

Note

Not all DTX Studio Clinic features are available in installations in an offline environment.

For detailed information on what is included, as well as guidance on specific tasks like updating the software or changing passwords, please refer to the DTX Studio Offline on <https://ifu.dtxstudio.com/>.